




SurgiField System Components

- A. Smart Control Module (SCM)
- B. Power button
- C. Tube connection Nozzle
- D. Tube connection ring (attached around nozzle)

- E. Pressure tube connection
- F. Battery housing
- G. SurgiBubble (SB) (Single Use)
- H. Arm Sleeves
- I. Material port with magnetic closure

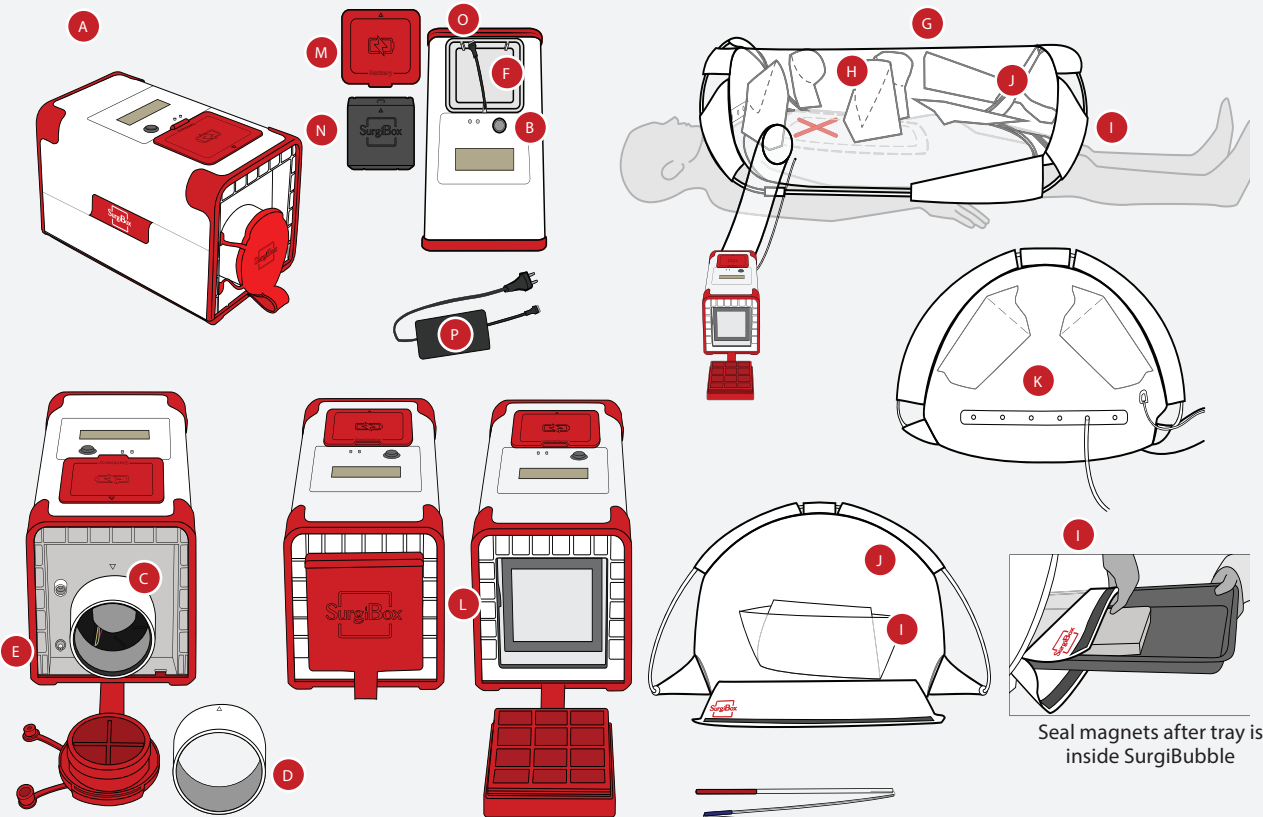

- J. Orthopedic sleeve
- K. Line ports
- L. Filter cap
- M. Battery cap
- N. Battery with white velcro strap

- O. Battery connection
- P. Charging cable with indicator lights
green = fully charged / red = charging
- Q. Pop-up frames (PuF) (2x)



1


Shave and clean the surgical site.

2

With the help of an assistant, assemble the Pop-up Frame (PuF) by snapping together the two halves then connecting them to form an oval.


3

Open package by peeling from corner tab. One corner is easier to peel than others.

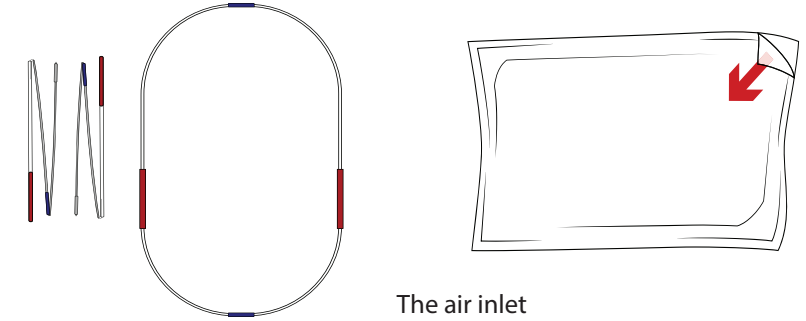


4

Orient the SurgiBubble (SB) and open over the patient. Remove the adhesive backing as opening. Firmly smooth adhesive around the surgical site so that there are no wrinkles/folds.

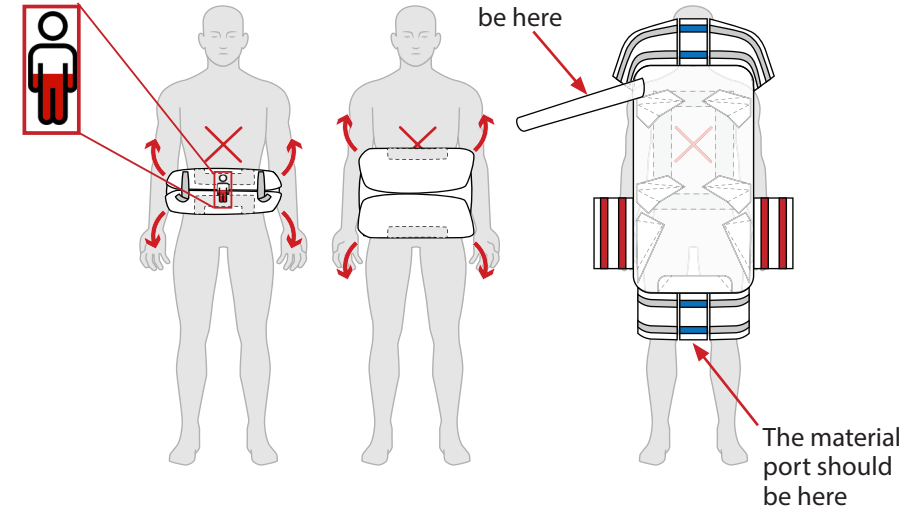


5

Place the Smart Control Module (SCM) inflation component alongside the SB at the head end. Do not attach the SCM yet. The set up should be 30"/75cm wide so the SCM is far enough from the SB that no kinking occurs.


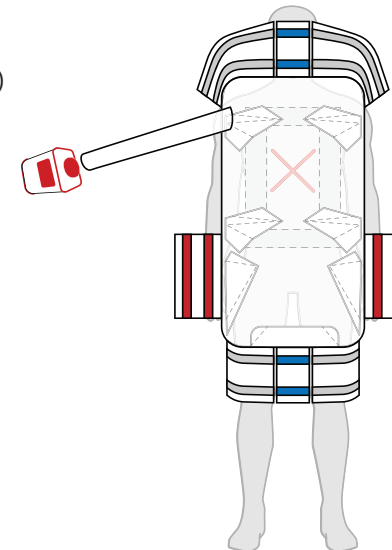


6


With the help of the white velcro strap insert and connect the battery, aligning the battery plug-in port with the arrow.

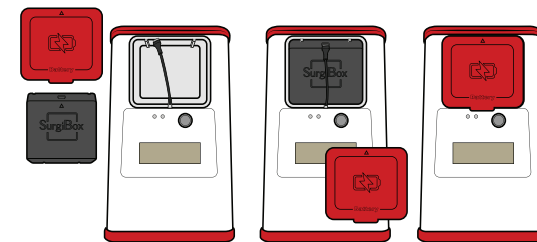


7 Attach the PuF to the SB following the colored labels on the PuF: **RED**, then **BLUE**.

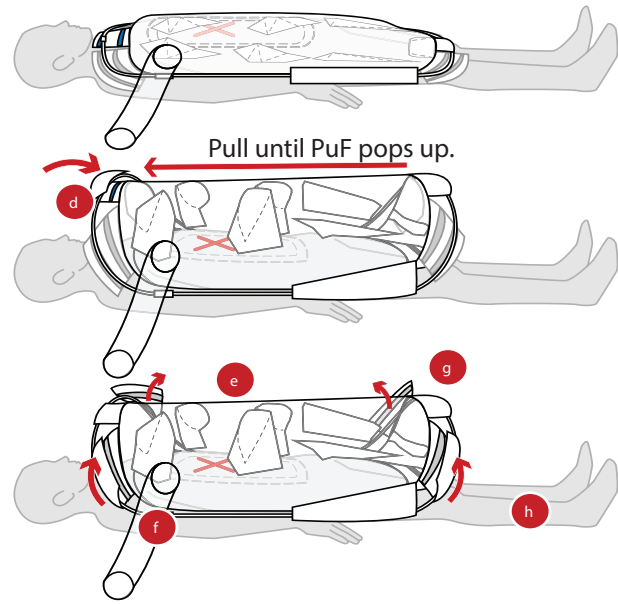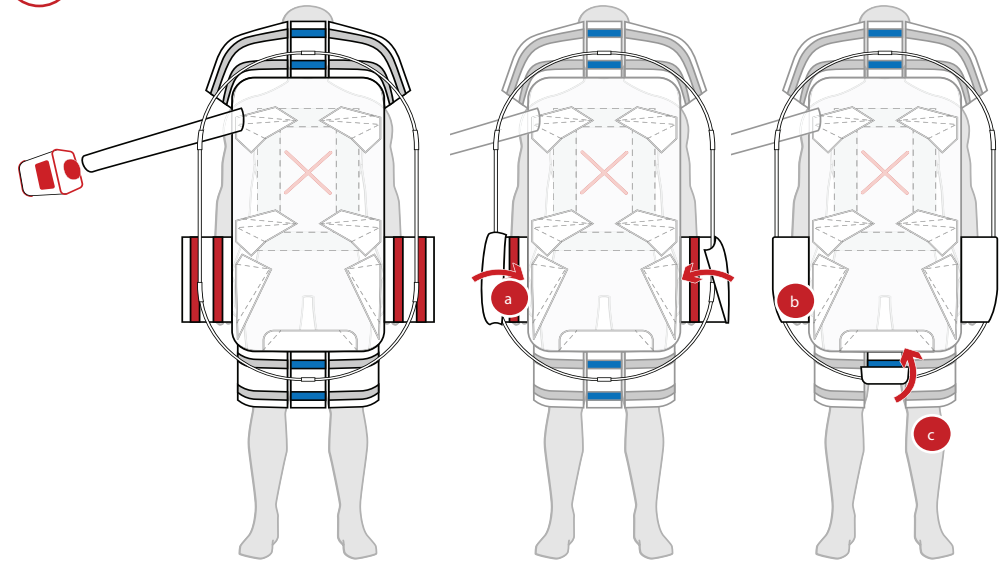


8 Open the nozzle caps, remove the connection ring from the nozzle, and attach the SB tube to the SCM as shown below.

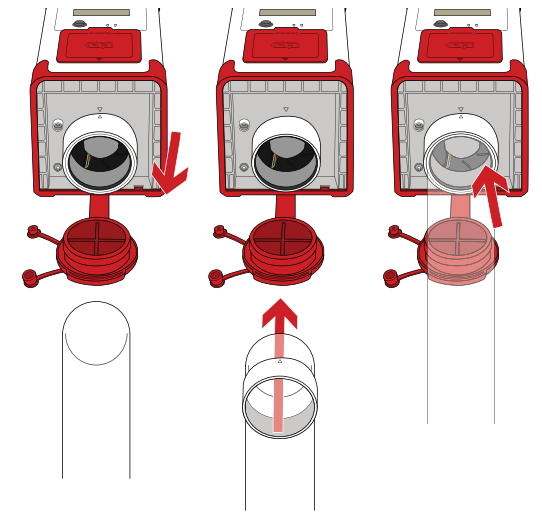

9 Connect the pressure tube to the lower port on the SCM. Test the connection of pressure tube and ring/nozzle by gently pulling on to them.

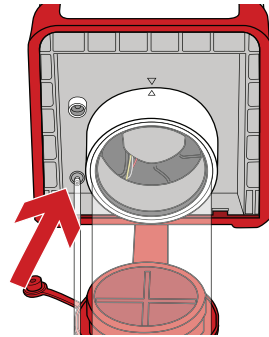


10 Remove filter cap on the back of the SCM.

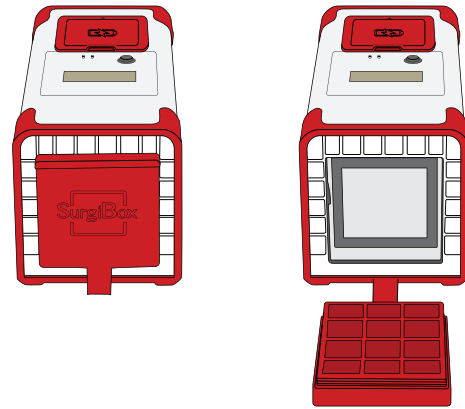


11 Power on the SCM to inflate the SB, wait until the SB has fully inflated.


12 Insert the surgical tray through the magnetic ports and ensure that the port is correctly sealed (no leaks) after closing it.

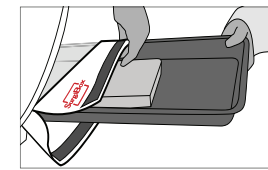

13 Be sure to be gloved BEFORE entering the SB. Unfold only the sleeves that will be used. Place thumbs through the thumb hooks in the sleeves, then double glove once inside the SB.

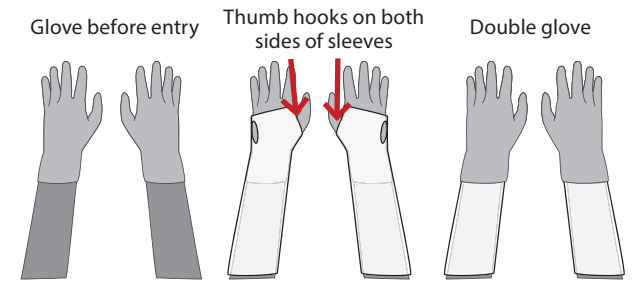


14 Remove the perforated panel of the SB to gain access to the surgical site.

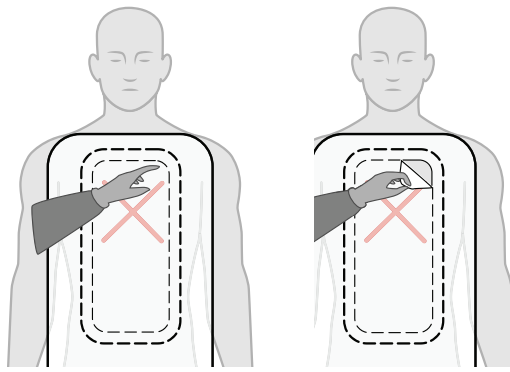

Gently press a desired corner then peel back.

15 After finishing all sutures, remove all tools, and suction excess fluids, then remove the SB.

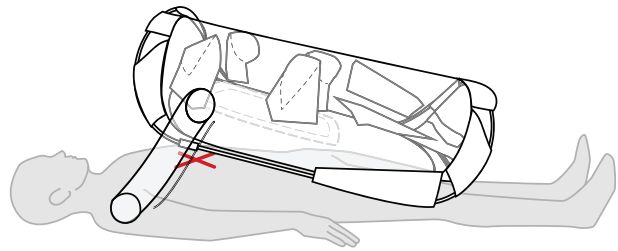

16 Clean the SCM and PuF using bleach wipes after use.

